



compass

Fundus Automated Perimetry

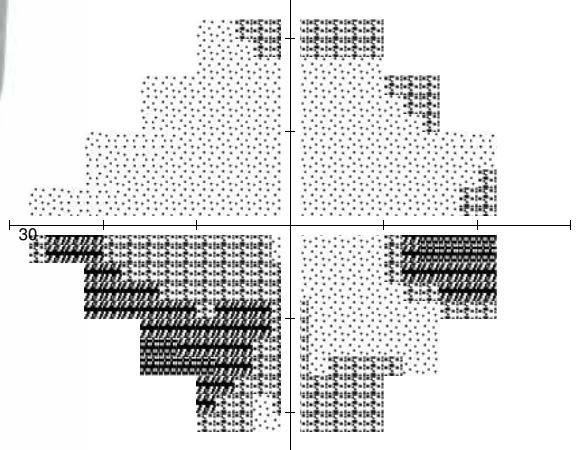

RNFL Detail

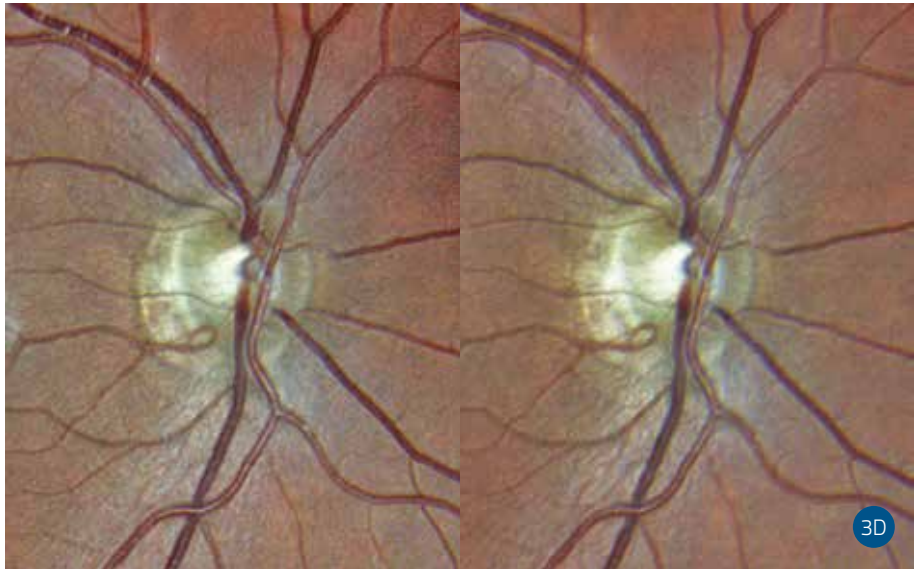

Stereo Pair



centervue

www.centervue.com
info@centervue.com

REV02-170626

How to Read Printouts

COMPASS combines in one page the standard deviation maps typical of SAP with structure-function correlation data as well as fixation analysis.

Legend

- 1 Patient info
- 2 Examined eye
- 3 Exam info
- 4 Mean Deviation, Pattern Standard Deviation and Fundus Perimetry Deviation Index
- 5 Fundus Perimetry (dB) over red-free image
- 6 Color image of ONH
- 7 Mean Deviation cluster
- 8 Standard VF map
- 9 Deviation maps
- 10 Fixation graph
- 11 GSS2

