

19 MAINTENANCE AND CLEANING PROCEDURE



WARNING: All maintenance activities can only be made when the device is turned off and unplugged from power supply socket.



WARNING: There are no user serviceable parts inside the device. Any covers can be removed only by authorized service staff.



WARNING: The main lens of the device should not contact the patient's eye or face.



NOTICE: It is not allowed to make any modifications of the SOCT.

During the whole period of use, SOCT does not need any special maintenance procedures. The outside surfaces of device should be kept tidy and free of dust and cleaned using mild cleaning solutions.

Take care not to get water or any other liquid inside the device. For hygienic reasons, after each examination the chinrest and forehead support should be disinfected.

19.1 Routine Cleaning

a) Cleaning of the device casing and equipment of the ME System (PC, monitor LCD, printer, keyboard and mouse PC)

Clean the casing with soft cloth. Use only cleaners dedicated for cleaning of electronic equipment. Clean periodically or when needed.

b) Cleaning of the applied parts (forehead and chinrest support)

The applied parts should be disinfected after examination of each and every patient. Use soft cloth, wet with ethanol solution for disinfecting. The applied parts are to be disinfected also after longer period of no use.

c) Cleaning of the lens

The lens should be cleaned sporadically, when contamination of lens surface is observable. For lens cleaning, use only cleaners applied in optics, that do not cause scratching of the lens. Clean the lens softly with little force. It comes with a lens cap to cover it when not in use. You should clean it to remove dust and oily smudges sporadically to ensure true tomogram images are obtained. You may use an alcohol prep swab or cotton swab dipped in isopropyl alcohol. Wipe dry with a soft, non-lining cloth or tissue.